



Protective surgical wear

It's in your hands – Infection prevention in veterinary medicine



Surgical scrubs



Before entering the operating area, disinfect hands and put on scrubs, cap, and mask.



In preparation for surgery, wipe-disinfect the work surface and have surgical gown and gloves at the ready. Make sure that they do not come into direct contact with the skin or the work surface.



Perform surgical hand washing (before operation or after leaving the OR area between procedures) and surgical hand disinfection.



Open the sterile primary packaging without touching the gown.



After the exposure time has elapsed, use the sterile cloth included in the package for final drying.



The gowns are folded such that the inside faces out. Slip your fingers into the sleeves on either side and step away from the table.

Surgical scrubs



The gown should unfold slowly downward.



Once it has completely unfolded, slide your arms in further.



Keep your hands inside the gown at all times.



Have someone else grasp the inside of the gown, pull it over your shoulders, and close the coat at your neck.



Have the other person tie the two additional bands on the inside of the gown.




Do not close the waistband until donning sterile gloves.



Remove the band from the green card and hold it secure.



Pass the green part to the assistant.

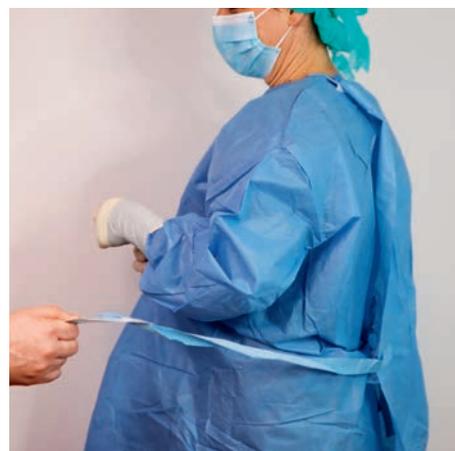

And then rotate carefully around.

Surgical scrubs



After that, pull the tape out of the paper loop.



Tie the two bands into a bow at the side.



You are now ready for surgery.



Additional information, this poster, and much more can be found at:
www.bbraun-vetcare.com/inyourhands

B. Braun Vet Care GmbH | Am Aesculap-Platz | 78532 Tuttlingen | info@bbraun-vetcare.de | www.bbraun-vetcare.com

Thank you very much – The pictures were taken in Hergolding with the generous support of the small animal specialists Frontier GmbH.

Subject to technical changes. This brochure may only be used as information about our products.
Reproduction in any form, partial or otherwise, is not permitted.